

The *Advanced* NPWT Solution™

Introducing the all new PRO-III™

Simpler, lighter, plus new features
and functionality

*Prospera® offers a complete
line of pumps, disposable
wound dressing kits,
collection canisters, and an
assortment of silicone drains.*



The PRO-III™ is the next step forward for Prospera's PRO series NPWT pumps. Get all the features and benefits of the Prospera operating system with advanced new features developed to provide enhanced safety and further simplify use.



New Features

- Larger display screen
- Color coded display screens to signal: pump idle (yellow), pump running (green) and error message (red)
- Lighter weight
- The proprietary Prospera Dual Filtration System* DFS™ is an industry first featuring internal and external bacteria filters to ensure patient safety and prevent cross contamination.
- Patient mode which will provide ability to:
 - Adjust alarm sensitivities
 - Monitor patient compliance

With all the existing features and benefits you have come to expect:

- Variable (VPT®) and Continuous (CPT) Pressure Therapy options
- Totally customizable
- Simple, comfortable dressing protocol
- Clinical support
- Virtually silent operation
- Cost-effective outcomes

Indications for Use

The PRO-III Negative Pressure Wound Therapy System is indicated for patients who would benefit from a suction device particularly as the device may promote wound healing.

Contraindications

When used for wound healing, the PRO-III is contraindicated in the presence of:

- Necrotic tissue
- Unexplored or non-enteric fistulas
- Untreated osteomyelitis
- Wounds containing malignant tissue
- Exposed arteries, veins, or organs

Precautions

Precautions should be taken in the presence of:

- Anticoagulation or active bleeding
- Difficult wound hemostasis
- Close proximity of blood vessels, organs, muscle, and fascia requiring adequate protection
- Irradiated vessels and tissue
- Bony fragments
- Untreated malnutrition
- Non-compliance

Prospera®

2831 Bledsoe Street
 Fort Worth, TX 76107
 P: 888-408-NPWT (6798)
 F: 888-674-NPWT (6798)

Technical Data

Air-flow rate of pump.....	8 liters/min
Negative pressure.....	Max.—200mmHg (Conversion factor: 1kPa ~ 7.5mmHg)
Collection canister.....	Disposable canister systems
Suction tubing.....	Various drainage systems, depending on supplier and specific application. Type and kind of use are determined by the medical staff.
Nominal voltage of power	
supply adapter.....	100–240V primary, 12VDC secondary
Power requirements.....	100–240V, 50/60Hz, 45W
Maximum load current.....	1.25A
Mains frequency	
of power supply adapter.....	50/60Hz
Nominal voltage of electronic	
circuit board.....	12V
Power consumption.....	15W (charging operation)/10W (charging only)
Current consumption.....	1.25A
Rechargeable battery.....	7.4V, 4.4Ah – lithium-ion battery
Charging time of empty battery.....	6 – 7 hours
half empty battery.....	3 – 3.5 hours
Dimensions (W x H x D).....	290 x 259 + 100 (canister) x 130mm (11.4 x 10.2 + 3.9 (canister) x 5.1 inches)
Weight (basic unit).....	2.2 kg (4.84 lbs)
Operating time.....	
AC operation: continuous operation	
DC operation: approx. 24 – 48 hours, depending on use	
Degree of protection	
according to IEC 601-1.....	Type BF
Risk class according to 93/42/EEC, IX.....	Ila
Sound emission.....	35dB (A)
Protection class according to IEC 601-1... II	

Email: info@prospera-npwt.com
 www.prospera-npwt.com